
Aquamantys™ System Cart


Assembly Instructions

Thank you for purchasing the Aquamantys™ System Cart. Before beginning the assembly of this cart, please verify that all of the parts listed below were included with your shipment. It is also helpful to review the assembly instructions in their entirety before beginning to assemble the cart.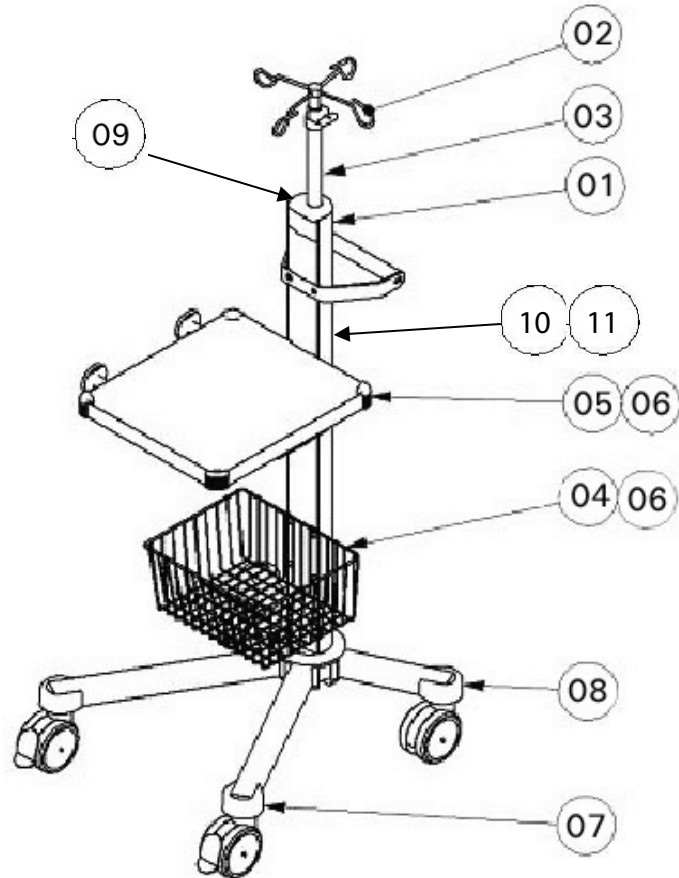

Two special tools have been included with your Aquamantys System Cart to ease assembly. This includes; 1) a large Allen wrench for tightening the hexagonal screws which are used to attach the castor/stabilizers to the cart, and the basket and the shelf to the support column; 2) a small Allen wrench for loosening and tightening the set screws for the infusion stand. No additional tools are required.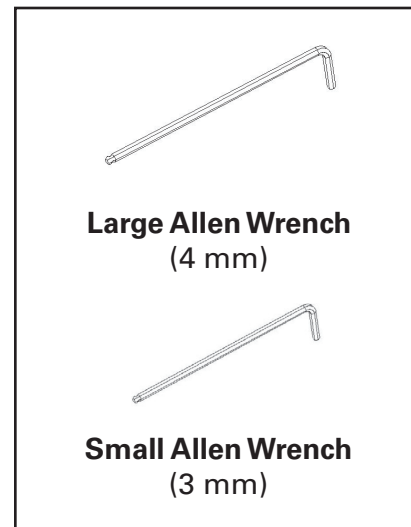


Parts List for Aquamantys System Cart (Catalog #30-401-1)

01 - Support Column (with attached handle)	Qty 1
02 - I.V. bag holder	Qty 1
03 - Infusion stand	Qty 1
04 - Basket	Qty 1
05 - Shelf	Qty 1
06 - Hexagonal screws (10 mm)	Qty 19 (one extra)
07 - Twin swivel castors/stabilizers with wheel locks	Qty 2
08 - Twin swivel castors/stabilizers without wheel locks	Qty 2
09 - End cap for support column	Qty 1
10 - Set Screws (6mm)	Qty 2
11 - Infusion stand clamps in Support Column (01)	Qty 2

Aquamantys System Cart Assembly Instructions


Step 1

Lay Support Column (01) horizontally on a hard surface with handle facing up. Attach the shorter length stabilizers which have the castors without wheel locks (08) to base of the hub by aligning holes in stabilizers with holes in stabilizer supports protruding from the base of the Support Column. Use two 10 mm hexagonal screws (06) per stabilizer. Tighten screws firmly with large Allen wrench.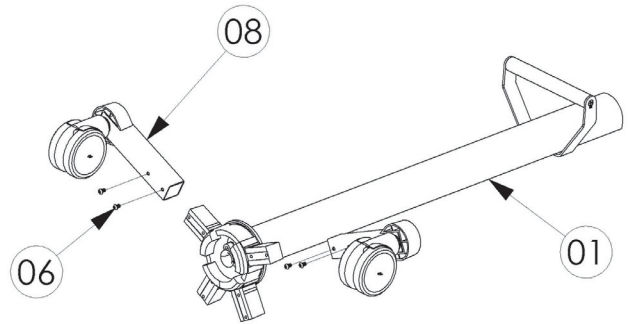


Step 2

Turn Support Column (01) over and lay horizontally on hard surface with handle facing down. Attach the longer stabilizers which have the castor with wheel locks (07) to base of hub by aligning holes in stabilizers with holes in stabilizer supports protruding from the base of the Support Column. Use two 10 mm hexagonal screws (06) per stabilizer. Tighten screws firmly with large Allen wrench.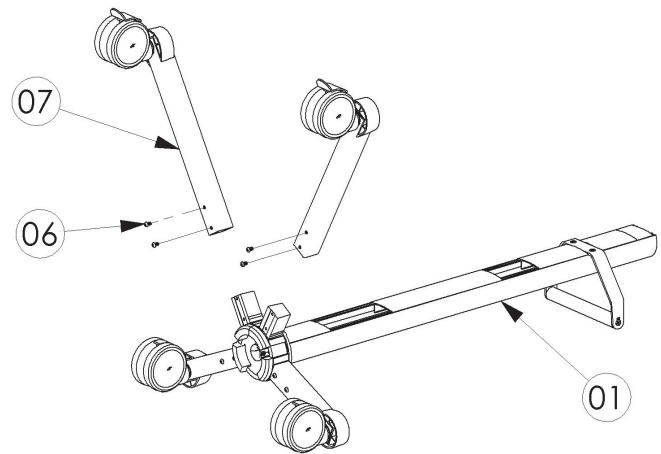


Step 3

Stand partially assembled cart up on its wheels and engage wheel locks. Loosen set screws (10) with small Allen wrench and insert infusion stand (03) through end cap center hole (09) then into the support column. Advance infusion stand through both infusion stand clamps (11) until the stand reaches the hub of the cart. Gentle pressure and a twisting motion may be required to advance the infusion stand fully. After the infusion stand is correctly positioned, tighten set screws (10) with small Allen wrench, then snap end cap (09) into support column.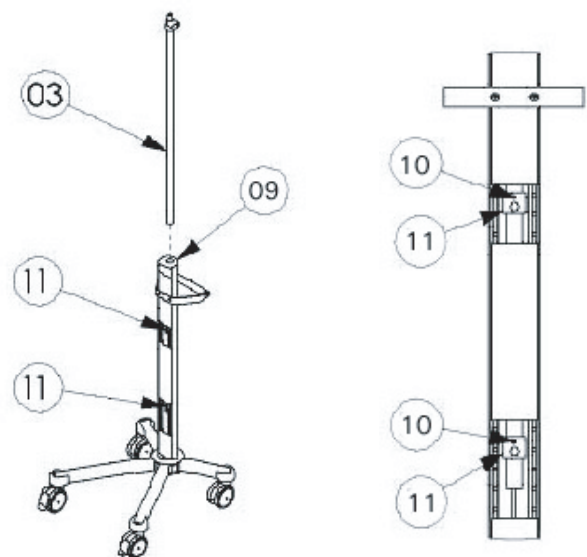


Step 4

Attach I.V. bag holder (02) to the top of infusion stand (03) by placing the center hub of the I.V. bag holder on top of infusion stand and then rotating it clockwise. Continue rotating clockwise until the I.V. bag holder is tightly attached.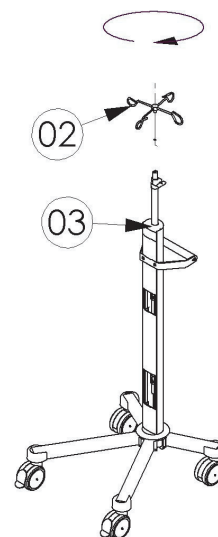

Step 5


Lay partially assembled cart on hard surface horizontally with handle facing down. Attach basket (04) to Support Column using four 10 mm hexagonal screws (06). Tighten screws firmly with large Allen wrench.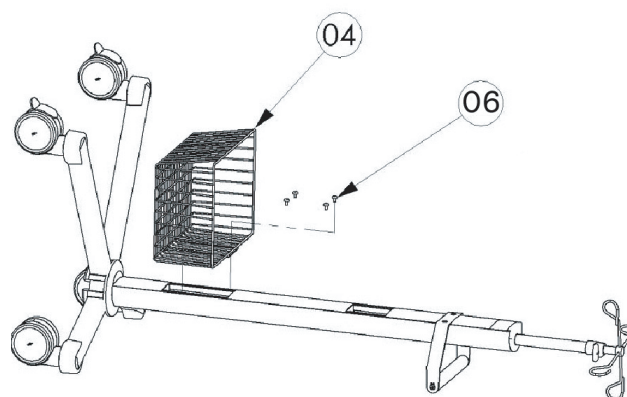

Step 6

Attach shelf (05) to Support Column using six 10 mm hexagonal screws (06). Tighten screws firmly with large Allen wrench.

Stand cart on its wheels. Cart is now assembled and ready for use.

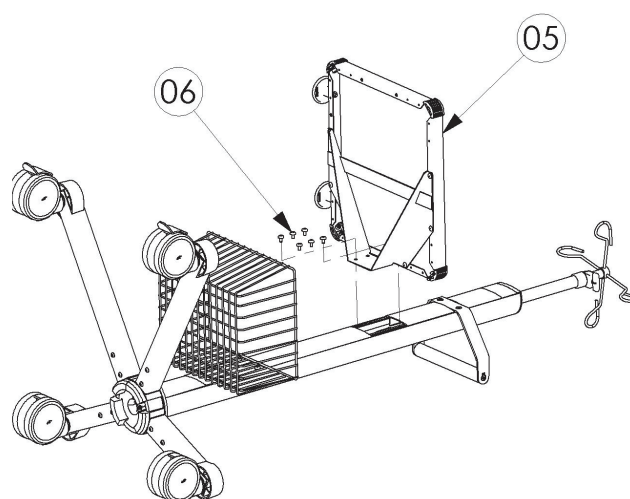Questions? Contact Salient Surgical Customer Service at 866.777.9400

Aquamantys™ System Cart

LIMITED EXPRESS WARRANTY

For one (1) year from the date of shipment from Salient Surgical, if an Aquamantys System Cart is found, to Salient Surgical's satisfaction, to be inoperable during normal and proper use in accordance with applicable instructions, Salient Surgical Technologies, Inc. will repair or replace the product, at its sole option, provided the product is returned, freight prepaid, in accordance with all return packaging and shipping instructions. A product repaired or replaced under this warranty will be warranted for the remainder of the original warranty period.

SALIENT SURGICAL TECHNOLOGIES, INC. MAKES NO OTHER WARRANTIES WITH RESPECT TO THE PRODUCT AND EXPRESSLY DISCLAIMS ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER. IN NO EVENT SHALL SALIENT SURGICAL TECHNOLOGIES, INC. BE LIABLE FOR ANY CONSEQUENTIAL DAMAGES.

THE ABOVE WARRANTY IS VOID ON ANY PRODUCT WHICH HAS BEEN MODIFIED OR REPAIRED OTHER THAN BY SALIENT SURGICAL OR AN AUTHORIZED REPRESENTATIVE, IMPROPERLY INSTALLED, USED, MAINTAINED OR STORED, OR SUBJECT TO ABUSE, MISUSE, NEGLIGENCE OR ACCIDENT. SALIENT SURGICAL IS NOT RESPONSIBLE FOR DAMAGE OR ANY OTHER LOSS DURING RETURN SHIPMENT.

Salient Surgical Technologies, Inc.
180 International Drive
Portsmouth, NH 03801
USA
www.salientsurgical.com

Customer Service:
Tel: 866.777.9400
Fax: 866.222.0900

Outside the US:
+1.603.742.1515
+1.603.742.1488

UK Telephone numbers:
Tel: 0808.101.1727
Fax: 0808.101.1726

© Copyright 2005-2009 Salient Surgical Technologies, Inc. All rights reserved. Printed in USA. SALIENT the SALIENT logo, AQUAMANTYS, and the AQUAMANTYS logo are trademarks of Salient Surgical Technologies, Inc. Additional trademarks are the property of their respective owners.

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© Copyright 2005-2009 Salient Surgical Technologies, Inc., tous droits réservés. Imprimé aux États-Unis. SALIENT, le logo SALIENT, AQUAMANTYS et le logo AQUAMANTYS sont des marques de Salient Surgical Technologies, Inc. Les autres marques appartiennent à leur détenteur respectif.
© Copyright 2005–2009 Salient Surgical Technologies, Inc. Alle Rechte vorbehalten. Gedruckt in den USA. SALIENT das SALIENT-logo, AQUAMANTYS und das AQUAMANTYS-logo sind Marken von Salient Surgical Technologies, Inc. Weitere Marken sind das Eigentum ihrer jeweiligen Inhaber.
© Copyright 2005-2009 Salient Surgical Technologies, Inc. Reservados todos los derechos. Impreso en EE. UU. SALIENT el logotipo DE SALIENT, AQUAMANTYS y el logotipo DE AQUAMANTYS son marcas registradas de Salient Surgical Technologies, Inc. Las demás marcas comerciales pertenecen a sus respectivos propietarios.
© Copyright 2005-2009 Salient Surgical Technologies, Inc. Tutti i diritti riservati. Stampato negli Stati Uniti. SALIENT il logo SALIENT, AQUAMANTYS e il logo AQUAMANTYS sono marchi registrati di Salient Surgical Technologies, Inc. Altri marchi registrati sono di proprietà dei rispettivi proprietari.
© Copyright 2005-2009 Salient Surgical Technologies, Inc. Todos os direitos reservados. Impresso nos EUA. SALIENT o logo DA SALIENT, AQUAMANTYS e o logo AQUAMANTYS são marcas registradas da Salient Surgical Technologies, Inc. As marcas registradas adicionais são de propriedade de seus respectivos donos.
© Copyright 2005-2009 Salient Surgical Technologies, Inc., alle rechten voorbehouden. Gedrukt in de VS. SALIENT het SALIENT-logo, AQUAMANTYS en het AQUAMANTYS-logo zijn handelsmerken van Salient Surgical Technologies, Inc. Aanvullende handelsmerken zijn in het bezit van hun respectieve eigenaren.